AMBULATORY LONG-TERM BLOOD PRESSURE MEASUREMENT

TRADITION MEETS INNOVATION

ABPMpro



Traditional Continuous

Oscillometric measurement PTT-based method




OSCILLOMETRIC BLOOD PRESSURE

BASIC DEVICE ABPMpro Oscillometric 24-hour Blood Pressure Measurement

• Simple application

- The main device slides directly on to the cuff no tube needed
- High patient comfort
- Internal ECG continuous recording of heart rate, independent of inflation process
 Allows for the detection of physical and psychological stress
- Internal sensor for activity and body position
 - Correlation of blood pressure to activity (during the day) and body position (at night)
 - Sleep-wake-estimation, precise classification of dipping / non-dipping
- Measurement during inflation and/or deflation
 - Back-up measurement during deflation if some artefacts occur during inflation
 - Increased information due to twice the number of measurement values
- Graphical display of inflation/deflation process in the software

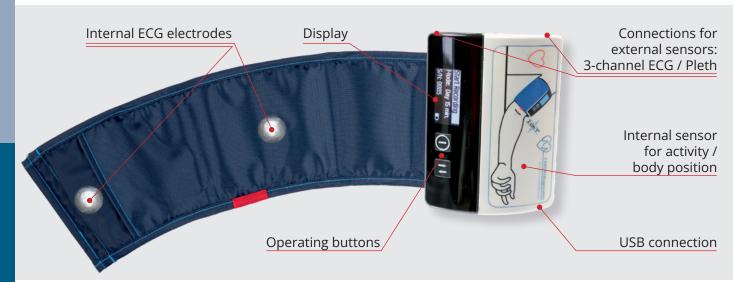

UPGRADEABLE

<u>3-channel ECG sensor (Holter)</u>

- Integrated sensor for precise determination of body position (thorax)
- Impedance cardiography for determination of – Breathing frequency
 - (Central apneas / Cheyne-Stokes breathing)
 - Pre-ejection period (PEP)
- Alternatively: pacemaker detection

Plethysmography (Pleth) sensor

- Detection of pulse wave
- Pulse wave analysis (PWA)
- Detection of autonomous arousal based on the pleth signal

When using both sensors:

Continuous, non-invasive blood pressure measurement WITHOUT inflation of the cuff based on the pulse transit time (PTT)


- Systolic / diastolic blood pressure values (in mmHg), beat-to-beat
- Heartbeat-precise representation of the blood pressure curve with all minima and maxima
- Detection of NBPF[™] (Nocturnal Blood Pressure Fluctuations)
- No falsification of the measured values no disturbance of sleep, no resulting blood pressure increase

Calculation of the pulse wave velocity (PWV) as a measure of arterial stiffness

Flexible blood pressure measurement methods

Individual choice of measurement method or combination of both methods (oscillometric / PTT)

SOFTWARE

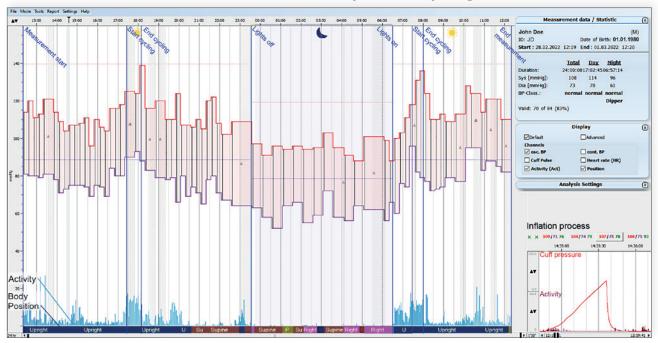

Automatic data transfer (USB connection), analysis and report generation

• Standard view: analysed data for a quick overview of the 24-hour blood pressure


or verify the values measured

Advanced view: raw data to check for artefacts

- Display of all measured values: sys/dia blood pressure, mean arterial pressure, heart rate, pulse pressure, body position, activity, day/night
- Display of the inflation/deflation process

REPORT

٠

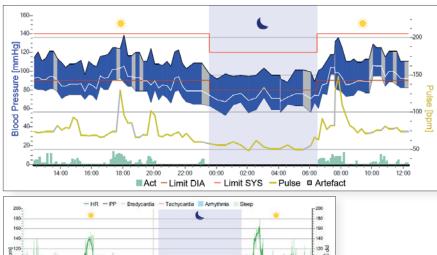




PTT BP Day/Night

						Day report					
	⊘ Min.	Aver.	⊘ Max.	SD	> Limit		Ø Min.	Aver.	Ø Max.	SD	> Limit
SYS [mmHg]	77	108	155	11,7	0,7 %	SYS [mmHg]	77	114	155	9	1,1 %
DIA [mmHg]	19	71	120	12.1	6,3 %	DIA [mmHg]	19	77	120	10	9,4 %
HR [bpm]	46	76	181	24.9	32.9 %	HR [bpm]	55	84	181	23.4	45.0 %
MAP [mmHg]	46	83	128	11,4	3,3 %	MAP [mmHg]	46	89	128	8,8	5,0 %
PP (mmHg)	20	37	105	7.7	1,2 %	PP [mmHg]	20	37	105	8,1	1,4 %
Ø Min and Ø Max o	omputed as m	ean value fro	m 8 values.			Limit SYS - 140mm	Hg, DIA - 50	mmHg, Puloc	- 80bpm, MA	P - 105 mml	ig, pp = 60 r
No. BP Values	81503 /8/	(% TPT)				Night report	_				
		· / IIII)					Ø Min.	Aver.	Ø Max.	SD	> Limit
Day/Night Dip			-			SYS [mmHa]	78	95	109	4	0.0 %
SYS [mmHg]	-16,7		O Reve	rse Dipper	r	SYS (mmHg) DIA [mmHg]	78	95 59	109 80	4	
SYS [mmHg] DIA [mmHg]	-16,7 -23,4	%	O Reve		r	DIA [mmHg]			80	6,4	0.0 %
SYS (mmHg) DIA (mmHg) HR (bpm)	-16,7 -23,4 -33,3	% %	<u> </u>	Dipper	r	DIA [mmHg] HR [bpm]	30	59		6,4 6,8	0.0 %
SYS [mmHg] DIA [mmHg]	-16,7 -23,4 -33,3 -20,2	% % %	O Non-I	Dipper		DIA [mmHg] HR [bpm] MAP [mmHg]	30 46 49	59 56 71	80 101 87	6,4 6,8 4,9	0.0 % 3.3 % 0.0 %
SYS (mmHg) DIA (mmHg) HR (bpm)	-16,7 -23,4 -33,3	% % %	O Non-I	Dipper		DIA [mmHg] HR [bpm] MAP [mmHg] PP [mmHg]	30 46 49 20	59 56 71 36	80 101 87 68	6,4 6,8 4,9 6,2	0.0 % 3.3 % 0.0 % 0.7 %
SYS [mmHg] DIA [mmHg] HR [bpm] MAP [mmHg]	-16,7 -23,4 -33,3 -20,2 -2,7	% % %	Non-I Non-I Dippe Extre	Dipper Ir me Dipper	r	DIA [mmHg] HR [bpm] MAP [mmHg] PP [mmHg] Limit 5Y5 = 120mm	30 46 49 20	59 56 71 36	80 101 87 68	6,4 6,8 4,9 6,2	0.0 % 3.3 % 0.0 % 0.7 %
SYS [mmHg] DIA [mmHg] HR [bpm] MAP [mmHg] PP [mmHg]	-16,7 -23,4 -33,3 -20,2 -2,7 of BP Le	% % %	Non-I Non-I Dippe Extre	Dipper Ir me Dipper	TESH (WHO	DIA [mmHg] HR [bpm] MAP [mmHg] PP [mmHg] Limit 5Y5 = 120mm	30 46 49 20 Hg. DIA = 60	59 56 71 36	80 101 87 68 - 70tpm, MA	6,4 6,8 4,9 6,2	0.0 % 3.3 % 0.0 % 0.7 % 2. PP = 50 m

- Individually adjustable reports
- Tabular or graphical presentation of measured data
- 24-hour overview of all blood pressure and heart rate values with classification of day and night
- Conventional reports with blood pressure and heart rate values in 15 or 30 minute intervals

SPECIFICATIONS

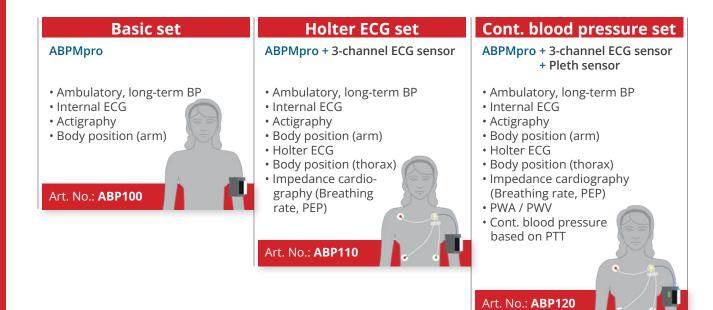



The ABPMpro is a compact and easy-to-use, long-term blood pressure monitor that provides higher diagnostic value by simultaneously recording actigraphy, body position and heart rate.

TECHNICAL DETAILS

Measurement intervals	3, 5, 10, 15, 20, 30, 60 min. programmable, sound signal before measurement start					
Measured pressure range	Systolic 60 - 230 mmHg, diastolic 40 - 130 mmHg					
Cuff sizes	S (18 - 24 cm), M (24 - 34 cm), L (34 - 46 cm)					
Cuff pressure	0 - 300 mmHg					
Display	Black / white, OLED					
Operating elements	2 operating buttons, multi-functional					
Power supply	Li-lon battery (1.800 mAh)					
Data storage /-transfer	8 GB micro SD, battery charging and data transfer via USB connection					
Data collection	16 - 24 bit signal resolution					
Size / weight (without cuff)	101 x 75 x 24 mm, weight: 125 g (incl. battery)					
Software	License free					
Interfaces	Connection per HL7 (optional)					
Included in delivery	Basic device ABPMpro, cuff size M, evaluation software, manual, USB connection and charging cable, measuring tape, bag					
Contact us for more information:						
	René van Liemt - LITRA-Distribution nl partner					



Contact us for more information: René van Liemt - LITRA-Distribution.nl partner E: info@litra-distribution.nl Tel. +31-(0) 6 - 51 817 280